


## RESPIRATORY

### OXYGEN DELIVERY DEVICES

#### Venturi Mask

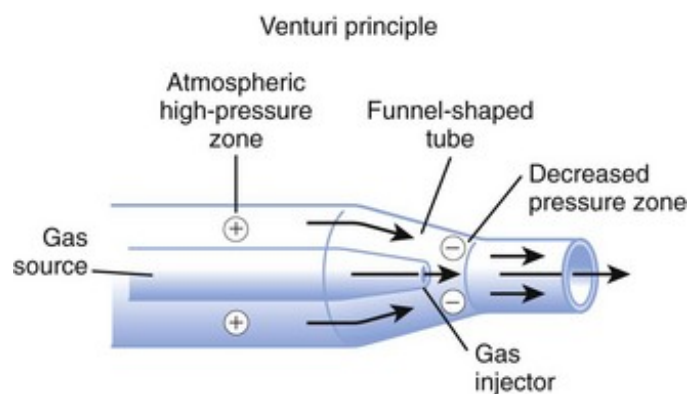

Venturi valve colour	Inspired oxygen concentration (%)	Oxygen flow (l/min)
Blue	24	2
White	28	4
Orange	31	6
Yellow	35	8
Red	40	10
Green	60	15



Mixes oxygen with room air creating high flow air with enriched oxygen at a predetermined concentration. Suitable for CO<sub>2</sub> retainers.

### Nasal cannula



Oxygen concentration depends oxygen flow rate:

1 L	24%
2L	28%
3L	32%
4L	36%

### Oxygen mask with reservoir



Oxygen concentration depends on oxygen flow rate:

6L	60%
7L	70%
8L	80%
9L	>80%
10L	>80%